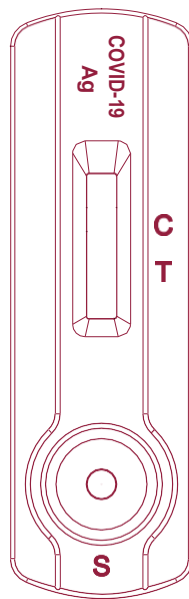
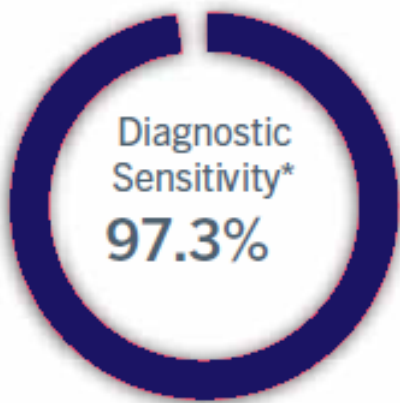
Coronavirus Ag Rapid Test Cassette (Swab)

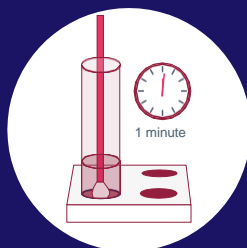
*ct value: 20 - 30

Rapid test for the detection of SARS-CoV-2 viral nucleoprotein antigens in human nasopharyngeal specimens


Sample

Extract

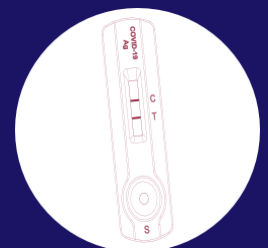Test

Result

The COVID-19 Rapid Antigen Point Of Care Test

The Coronavirus Ag Rapid Test Cassette (Swab) is an in vitro immunochromatographic assay for the qualitative detection of nucleocapsid protein antigen from SARS-CoV-2 in nasopharyngeal (NP) swab specimens directly or after the swabs have been added to viral transport media from individuals who are suspected of COVID-19 by their healthcare provider. It is intended to aid in the rapid diagnosis of SARS-CoV-2 infections. The Coronavirus Ag Rapid Test Cassette (Swab) does not differentiate between SARS-CoV and SARS-CoV-2. This real-time test is a practical solution that is straightforward to implement, even at scale. It is intended for use as an aid in the diagnosis of COVID-19 infections, conducted by a healthcare professional, with results provided directly to patients within 15 minutes with SARS-CoV-2.

The Coronavirus Ag Rapid Test Cassette (Swab) has been evaluated with specimens obtained from patients. A commercialised molecular assay was used as the reference method. The results show that the Coronavirus Ag Rapid Test Cassette (Swab) has a high overall relative accuracy, as shown in the following table:

		RT-PCR		
		Positive	Negative	Total
Coronavirus Ag Rapid Test Cassette (Swab)	Positive	71	0	71
	Negative	2	130	132
	Total	73	130	203

Benefits at a glance

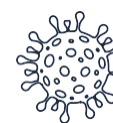
- ✓ High specificity
- ✓ High sensitivity
- ✓ Easy to use
- ✓ Fast and reliable test results in only 15 minutes
- ✓ Test can be performed using nasopharyngeal specimens
- ✓ Can be stored at room temperature
- ✓ All test components (including sterile swabs) are included
- ✓ Packaged in boxes of 20
- ✓ No additional equipment needed

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