

COVID-19 IgG/IgM Rapid Test Kit (Colloidal Gold)

IgG Results



IgM Results




Rapid test for the detection of SARS-CoV-2 antibodies in human blood, plasma or serum samples




Puncture



Collect

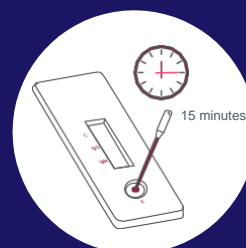

Test

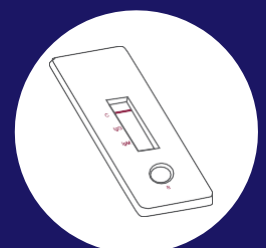

Result

The COVID-19 Rapid Antibody Point Of Care Test

The COVID-19 IgG/IgM Rapid Test Cassette (Whole Blood/Serum/Plasma) is a lateral flow immunoassay intended for the qualitative detection and differentiation of IgM and IgG antibodies to SARS-CoV-2 in human venous whole blood, plasma from anticoagulated blood (Li+ heparin, K2EDTA and sodium citrate), or serum. The COVID-19 IgG/IgM Rapid Test Cassette (Whole Blood/Serum/Plasma) is intended for use as an aid in identifying individuals with an adaptive immune response to SARS-CoV-2, indicating recent or prior infection.

The COVID-19 IgG/IgM Rapid Test Cassette (Whole Blood/Serum/Plasma) has been evaluated with clinical specimens whose status was confirmed using RT-PCR. The results are presented in the following table:

| | | RT-PCR | | | |
|--|----------|-------------|------------|------------|------------|
| | | Positive | Negative | Subtotal | |
| NADAL[®] COVID-19 IgG/IgM Test (IgG) | Positive | IgG+ / IgM+ | 78 | 0 | 78 |
| | | IgG- / IgM+ | 0 | 1 | 1 |
| | Negative | IgG+ / IgM- | 9 | 2 | 11 |
| | | IgG- / IgM- | 3 | 98 | 101 |
| Subtotal | | 90 | 101 | 191 | |

Benefits at a glance

- ✓ High specificity
- ✓ High sensitivity
- ✓ Easy to use
- ✓ Fast and reliable test results within 15 minutes
- ✓ Test can be performed using blood sample
- ✓ Can be stored at room temperature
- ✓ All test components (including sterile swabs) are included
- ✓ No crossreactivity with the specimens was observed when tested using the HEALGEN[®] COVID-19 IgG/IgM Test Cassette.




CE
Marked



US
Manufacturer



Immediate
Results

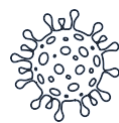

COVID-19
Specific



Deployable
at Scale



Designed
for HCPs

Tel: +96264622301

E-mail: info@bioclinicalme.com